




First Time in India MOST COST EFFECTIVE LAUGHING GAS UNIT






ANXIOLYSIS A level of sedation in which a person is very relaxed and may be awake. The person is able to answer questions and follow instructions. Anxiolysis is caused by special drugs and is used to help relieve anxiety during certain medical or surgical procedures also called minimal sedation.

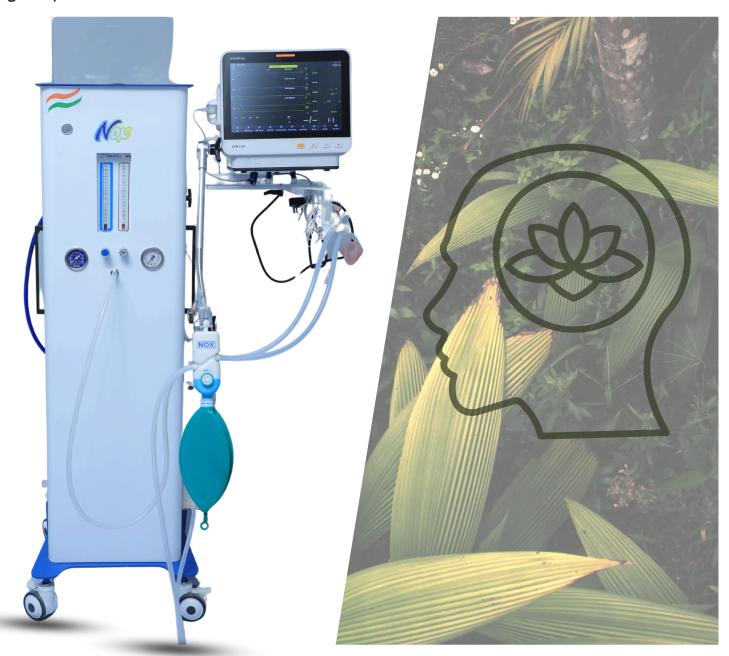

ANALGESIA Loss of sensation of pain that results from an interruption in the nervous system pathway between sense organ and brain.

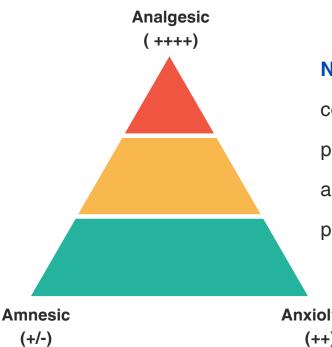

 (N_2O) Nitrous oxide at concentrations below 50% is primarily utilized as an analgesic and anxiolytic agent rather than as a primary sedative.

Anxiolytic (++)

WE CLAIM,

N2O IS MORE ANALGESIC THAN SEDATIVE.

IN DENTISTRY WE NEED ANALGESIA & ANXIOLYSIS NOT SEDATION. SO, WHY TO PROMOTE AN OXYMORON CONSCIOUS SEDATION. N2O OFFERS A CALMING SOLUTION FOR PATIENTS OF ALL AGES

NOX Sterility of Equipment

- 1. Circuit & mask both autoclavable
- 2. Only circuit which can be used with HME filter (antibacterial. anti viruses) to avoid Cross Contamination
- 3. Dry sterilization is also possible with alcohol wipes



4. ETO sterilization is also possible

Three Major Challenges in Dental Patient

1. Fear of Pain



2. Apprehension (fear of unknown)



3. Anxiety



N₂O easily manages this to make your Patient Comfortable and Pain - Free Experience.

NOX as a Day-Care Tool

- Excellent analgesia. (Pain-free experience)
- Titrable calmness (Anxiolysis)
- Fastest onset and Rapid recovery
- No- residual effect. (Immediate discharge of Patient is possible)
- Suitable for all age patient
- Lowest operational cost with NOX Machine
- Can be used with minimal pre-procedure instructions
- No major Contraindications can be used in highly morbid patient too

Fail-Proof Safety Features


We have integrated a wide range of features into our NOX systems to provide complete control for the healthcare team, thereby ensuring the utmost safety for patients undergoing procedures.

These features include:



OXYGEN- FAIL SAFE SYSTEM

In an oxygen supply failure, nitrous oxide administration will be stopped, as it requires oxygen.

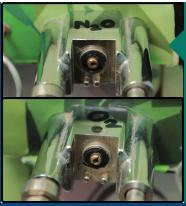

STANDARD JOINT OR GAS PIPELINE

a. Pin index SafetySystem (PISS)b. Diameter index SafetySystem (DISS)Customizable as per your need.

ADJUSTABLE PRESSURE LIMITING (APL VALVE)


This machine offers unique positive pressure ventilation for respiratory patients, a feature absent in other devices.



HYPOXIC GUARD

NOX is designed with an N₂O limiter.

The maximum amount of N_2O that NOX can deliver is 75%.

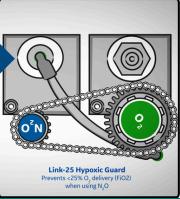




OXYGEN FLUSH

An integrated Oxygen flush button on the machine provides an immediate flow of 25 LPM of Oxygen to the patient.



AUDIBLE OXYGEN ALARM

When your oxygen pressure drops, an alarm will sound to alert you before cutting off the N₂O supply.

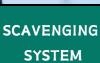

The most efficient scavenging system protects doctors and paramedics while minimizing CO₂ emissions.



NON-REBREATHING VALVE

The machine operates on a rebreathing principle, preventing CO₂ poisoning and cutting gas consumption by 50%.





Technical Specifications NOX ADVAITA

Oxygen	Maximum flow 10L/min.
N ₂ 0	Maximum flow 12L/min.
Dimension	(H*W*O)
Weight of machine	=50KG
Precisely calibrated flow	±100 ml/min
N ₂ O	0 - 75%
O ₂	25% - 100%
Oxygen Flush	(>25L / min)
Rebreathing bag. Sizeable ale to patent weight. to decrease gas wastage.	(0.5L/1L/2L/3L)
Adjustable mask size	(S/M/L)





- Hypoxic guard flow meter (ISO/CE certified)
- Useful in both Cylinder & central-supply
- Equipped with Scavenging System
- Equipped with all safety standard Safety mechanism
- Only machine with Storage
- Positionable rebreathing bag to monitor respiration at Consultant's eye level

NOX - Signature Aspects





Monitor Stand



Storage Compartment on Top



Rapid Titration Circuit



Audible Oxygen Alarm




Open Active Scavenging
System with CO2 Absorber



Adjustable Rebreathing Bag Arm

NOX - Applications

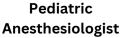

Cosmetic & Skin Department (Plastic Surgery)



Endoscopy



NOX - Product Variant





NOX - ADVAITA WITH POWDER COATING

NOX - URJA WITH VINYL COATING

Manufacturers of Medical & Healthcare Equipment's GST: 24AAICN5942R1ZR



AHMEDABAD Shop No 1, Hari Vila, A-Apartment, opp. Gujarat High Court, S.G. Highway, Sola, Ahmedabad, Gujarat, 380061

NAIROBI, KENYA
Office 14, Vision Plaza,3rd.
Fir.Mombasa Rd, P.O. Box
10016700101,
Nairobi, Kenya.

Connect us for more Details!

- +91 9898985736
- +91 99756 14527
- Noxanalgesia@gmail.com